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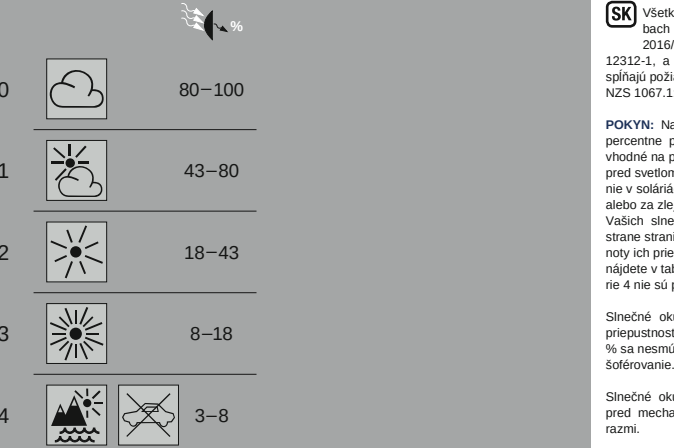
EN All sunglasses from Eschenbach meet the requirements of regulation (EU) 2016/425, 2017/745 and the DIN EN ISO 12312-1 standard, and consequently bear the CE marking. They also meet the ANSI Z80.3 (USA) and AS/NZS 1067.1:2016 (Australia/New Zealand) requirement standards.

PLEASE NOTE: Our sunglasses provide you with 100% protection from natural UV radiation. They are not suitable for looking directly into the sun, neither for protection from artificial light sources nor for use in solariums. Do not use in the dark or poor light conditions in road traffic.

You can find details of the filter category of your sunglasses on the inside of the temple. Please refer to the table in Fig. 1 for a description of the filter categories, their transmission values and recommended use.

Sunglasses in category 4 are not approved for use when driving.

Sunglasses and phototropic filters with a degree of light transmission of less than 75% in bright conditions must not be worn at dusk or at night in road traffic.

Sunglasses are not suitable for protecting the eyes from mechanical hazards such as impact.

Ensure that other people, especially children, are aware of these dangers!

CARE: Only use a soft glasses cloth for cleaning. Remove heavy dirt with water and a mild detergent. Do not use solvents such as alcohol or acetone! Preferably keep in a case. Do not use scratched or damaged glasses.

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